

# Rescue Procedure for Stretcher




Rig the Jason's Cradle Stretcher – unpack the unit and setup with orange flotation tubes outermost.



◀ **ORANGE FLOATS OUTSIDE** ▶

Attach the Jason's Cradle Stretcher to your deck crane or davit. Lower the stretcher into the water and manoeuvre the casualty into the unit. The frame of the stretcher will submerge whilst the floats keep the top of the unit at water level.

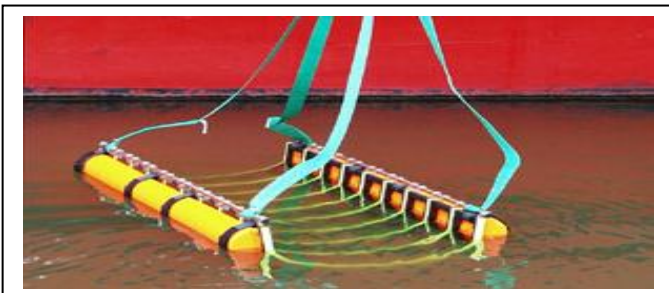

The unit can now be raised either manually or with a block and tackle or power assisted hoist. Raise the casualty steadily upwards and slowly onto the deck. Now that the casualty is out of danger they can be taken to a position of safety where first aid can be administered.

**Jason's Cradle® MOB systems**

**+44 (0) 1202 874365**

[www.jasonscradle.co.uk](http://www.jasonscradle.co.uk)

**SOLAS  
APPROVED**